

Also available from Jenx.....



Turtle



Tiger



Beta



Giraffe



Monkey



Toilet Seat



# NURSERY TABLE USER GUIDE



Size 1 - NR65  
Size 2 - NR66

We are delighted that you have chosen to use a Jenx therapeutic product for your child. Jenx products come in all sorts of shapes and sizes and each one is designed to offer a better quality of life to children by improving their posture and therefore enabling them to do more. When a child finds that they can do something, it encourages them to try other things. This is the basis of motivation and motivation is built in to each of our designs. Jenx Limited has been designing and making therapeutic products for children for over 20 years. If you wish to have more information about any products in the Jenx range, please contact Sales Services on the number below.



Jenx Ltd, Wardsend Road Sheffield S6 1RQ  
Tel: +44 (0)114 2853376 Fax: +44 (0)114 2853528  
Email: info@jenx.com Website: www.jenx.com

© 2004 Jenx Ltd

### IMPORTANT

**FOR ADVICE ON  
THE USE OF THIS  
PRODUCT PLEASE  
PHONE OUR  
AFTER SALES  
DEPARTMENT ON  
0114 285 3376**


### INDEX

Page 2.....	Important safeguards
Page 3.....	Making & using the adjustments

*Please read these instructions carefully before use.*

# IMPORTANT SAFEGUARDS

- Failure to follow instructions, especially with regard to moving parts may pose a risk to the user. Always read instructions before use.
- Assemble product fully before use. Please contact us if you have any problems at this stage.
- Look after your product well, clean regularly and do not use abrasive cleaners.
- All Jenx products are tested to current fire regulations. Always keep products away from naked flames and cigarettes.
- The Nursery Table should be used on a level floor and **never** on a raised surface.
- Always use all the safety straps provided.
- This product contains small parts which could present a choking hazard to small children.
- Do not use this product if you suspect it is faulty or broken. Contact us for advice.
- This product conforms to the European Medical Devices Directive. It has not been tested in conjunction with other manufacturers' products. Do not add any accessories other than Jenx approved accessories to this product. If in doubt, contact us for advice.
- The batch number is written on the fire label (see picture below) which is located underneath the tray. Please quote this number on all correspondence.

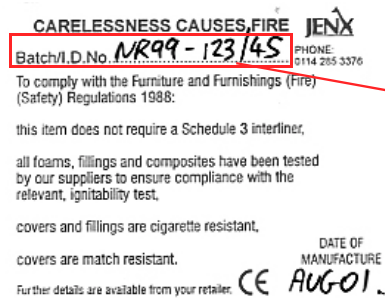

Batch number

Any service carried out by persons other than a Jenx authorised service person will invalidate the guarantee and Jenx can accept no liability for any injury or damage caused.

## Assembly

This product comes complete and requires no assembly.

### Making & using the adjustments

#### The table angle

##### HOW?

To adjust the table angle loosen the wingnuts as shown in fig 1. Adjust to desired angle and re-tighten.

##### WHY?

The table can be angled to provide an inclined surface with cut-out for your child or can be angled to provide an easel with ledge to prevent things falling off. With the surface inclined and a cut out, the Nursery Table is ideal for handwriting or drawing activities and a very small angle will make it easier for toys to continually fall back towards your child in play.

The Nursery Table as an easel means that your child can enjoy painting or other activities where an easel surface makes the job easier. Paper will stick easily to the surface with clear sticky tape, Blu-Tack® or similar.

#### The table height

##### HOW?

To adjust the table height, loosen the wingnuts as shown in fig 2. Set the desired height and then re-tighten.

##### WHY?

Table height will affect your child's ability to carry out the activities on offer. If the table is too low, it may be difficult for your child to use their hands well as their arms may be stretched already. If the table height is too high, your child may not have the power to keep their arms that high throughout an activity.

The ideal height for your child in any activity is the one where they look comfortable and can use their arms and hands at their best. This may differ from activity to activity.



Fig 1



Fig 2