


Jenx Junior 1, 2, 3



COMFORTABLE, CONTOURED
MODULAR SEATING SYSTEM!

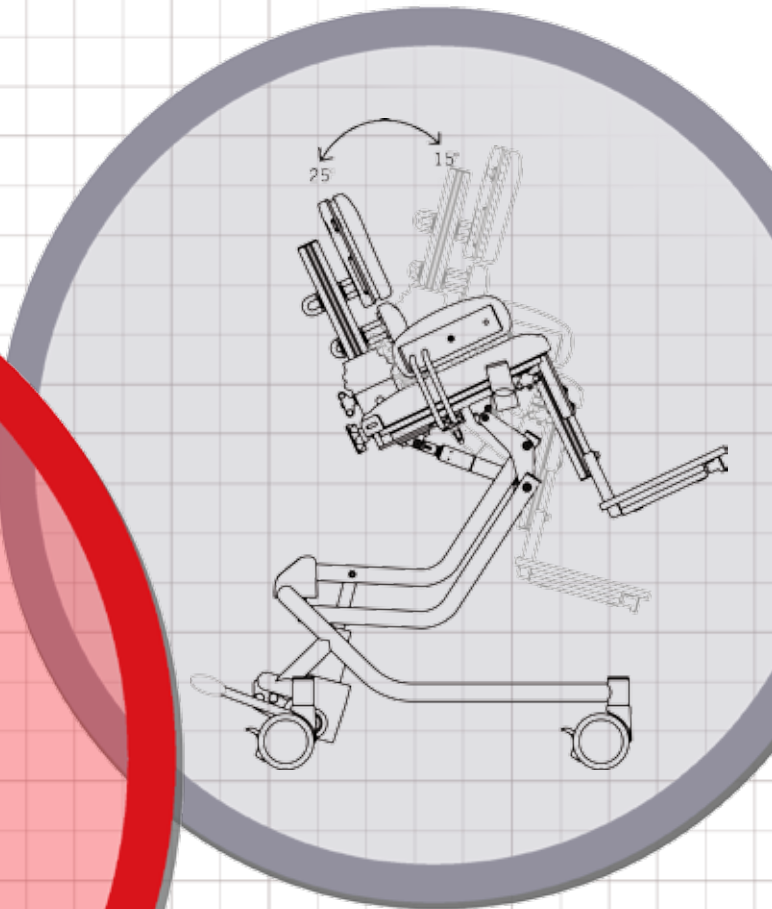

Jenx Junior 1, 2, 3



New Jenx Junior is a fully adjustable modular seating system that provides excellent postural support for a wide range of users, from 18 months to 12 years.

Jenx Junior has been designed around the importance of comfort for the user and the necessity to provide a stable hip base.

The new system has multiple adjustments and benefits from larger wing nuts for easier tool-free adjustment and a range of new accessories to compliment the existing range; the key feature being the ergonomically shaped seat pad and back pads; contoured and extremely comfortable with a soft, PU moulded base including targeted use of memory foam.

A removable abduction roll within the seat cushion, combined with contoured cushion sides and hip pads, provides greater hip and leg control.

An Ischial Bar option, when fitted to the memory foam pad, will also help to prevent the child from slipping forward into sacral sitting.

Jenx Junior is supplied with a Jenx 4-point lap strap as standard. This sits over the child's ASIS' and helps to maintain a firm and neutral hip position.


Jenx Junior is also supplied with one set of reversible seat and back pad covers chosen from a wide selection of colours and finishes plus a second seat pad cover as standard, enabling the system to be used when one set of covers is being laundered for example

..... easy as 1,2,3

Jenx Junior is a modular seating system in 3 sizes. All fittings can be specified at time of order.

- ✓ Easy for therapist to adjust for growth
- ✓ Seat height easy to adjust
- ✓ Pads easy to adjust
- ✓ 4 point lap strap easy to use
- ✓ Tilt-in-Space easy to operate
- ✓ Hydraulic or powered base
- ✓ Accessories easy to attach

SEAT PAD CROSS SECTION

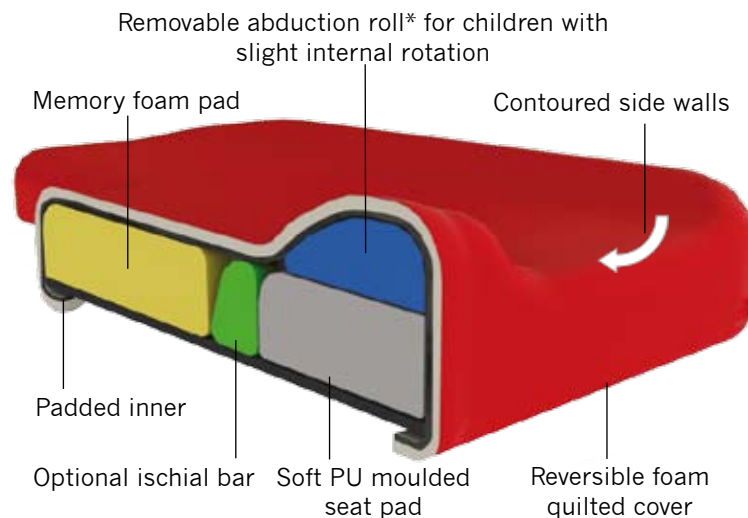

*Optional abduction block available for children with increased tone and tonal pull to adduction

Manufactured with 18 key product features



● WASHABLE, REVERSIBLE COVERS
Wide range of colour options plus a second seat pad cover supplied as standard.

● HEADREST OPTIONS
Jenx Multigrip, Oval, Memory Metal, Stealth and Savant headrests can be fitted.

● LAP STRAP BUCKLE HOLDERS
Tuck away lap straps for easy transfer, preventing lap strap from snagging under child.

● HIP PADS
Adjustable for width and angle, can be fitted with extensions.

● INTEGRAL ABDUCTION ROLL
To promote neutral leg position, removable PU for easy cleaning.

● CONTOURED SEAT PAD
Soft, easy clean PU with targeted memory foam insert.

● ABDUCTION BLOCK FITTING
Additional accessory for children with increased tone and tonal pull into adduction.



● 4 POINT LAP STRAP
Hold ASIS in correct position for secure hip base.

● SMALL FOOTPRINT
Jenx Y base with convenient flip-away foot pedal and safety knob. Fits through most standard doors.



Jenx Junior standard configuration: Y base with lockable castors, lockable tilt-in-space lever, flip-away foot pedal, seat with removable abduction roll and memory foam insert, 4 point lap strap, soft PU back pad, seat & back pad reversible covers, spare seat pad cover, angle adjustable soft PU lumbar roll, angle adjustable soft PU hip pads and 1 piece flip-up footplate.

and 6 major adjustable components

● CONTOURED BACK PAD

Soft, easy clean adjustable PU pad. Can be fitted with adjustable thoracics, elbow blocks, waistcoat harness or shoulder harness.

● BACK ANGLE

Close angle for children with extensor thrust or open angle for more support for children with low tone or for children who are not comfortable with 90/90 seating. Adjustable from 15° prone to 25° recline.

● LUMBAR & THORACIC SUPPORT

Soft, easy clean PU. Adjustable for height, angle and depth.

● TRAY ARMS

Firm fit, easy clean arms with large wing nuts for easy height and angle adjustment, with improved height adjustment lock.

● ISCHIAL BAR

Optional PU bar that helps to prevent child slipping into sacral sitting.

● EASY-CLEAN TRAY (not shown)

2 sizes available plus tray insert to reduce gap if required, grab bar, activity frame, tray padding, chest pad and elbow block accessories.

● TRANSLUCENT TRAY INSERT (not shown)

To hold and protect pictures and communication signs.

● 1 PIECE FOOTPLATE

Adjustable flip-up for easy transfer. Use with or without sandals. Accessories include individual footplates, padded box footplate, footplate pad.

● EASY ADJUSTMENT

Tool free or simple allen key adjustment, no need to dismantle or remove parts for access.

● Back Pad & Lumbar Roll

Undo large wing nut anti-clockwise and adjust height, reach and pad angle. Tighten by hand.



● Back Angle

To open or close the seat angle, loosen top wing nut by hand then turn large wheel for correct angle. Lock top wing nut.



● Seat Depth

Loosen wing nuts under seat front to move seat pad backwards or forwards. Tighten by hand.



● Locking Tilt-in-Space

Locate locking tilt-in-space lever under seat, unscrew lever and push to side. Adjust angle and screw lever back in to lock.



● Footplate Height

Loosen allen bolt with key supplied and adjust to correct height.




● Flip-away foot peddle


Press down to adjust height or lift up to lower. Push lever back to lock away.




ACCESSORIES & OPTIONS

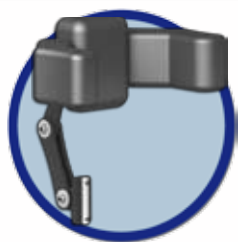

Jenx Multigrip® Headrest
Made with comfortable, easy to clean polyurethane padding (PU) and adjustable in 3 planes.

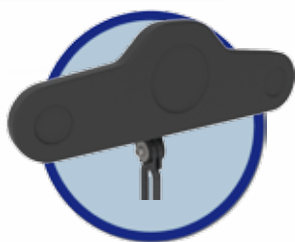

Jenx Memory Metal Headrest
Fully adjustable for custom fit. Provides a sub-occipital ledge which promotes a neutral head position.

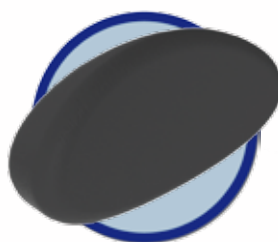

Jenx Oval Headrest
For occipital, sub-occipital minimal support or to act as a boundary.



Stealth & Savant
A wide range of headrests, prompts and switches can be fitted to Junior. Ask for information.




Tray Arms
Firm fixing and easy-to-use large wing nuts for angle and height adjustment.



Translucent Tray Infill
To protect pictures and signs placed underneath.



Tray - Reduced Cut-out
Infill for smaller children or to reduce gap.

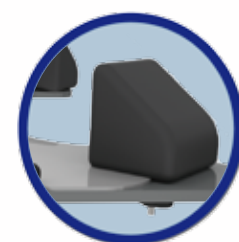

Elbow Blocks
For tray or seat back pad, prevents retraction and promotes mid-line working position.




Fixed Push Handle
With soft grips, secure and easy to adjust for height.




Folding Push Handle
With soft grips, secure and easy to adjust for height, angle or fold away.



Flip-away Thoracics
Short or long for extra support, with or without de-rotational chest strap.

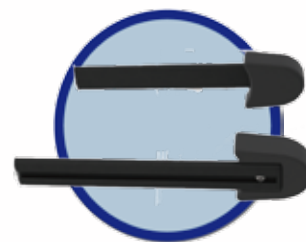


Hip Pad Extensions
To prevent abduction (frogging) position. Easy to use and adjust.

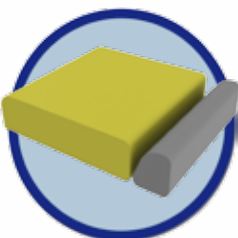

Ischial Bar
Helps to block negative pelvic position, rotation or sacral sitting.



Waistcoat or Shoulder Harness
A comfortable and non-intrusive vest for added safety.

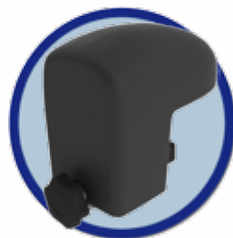

Abduction Block
For children who have a strong tonal pull to adduction. 3 sizes.



Powered Base
Hand held actuator for simple high-low adjustment.

Accessories not shown: Tray with edging * Full tray padding * Tray grab bar * Tray activity frame * Fixed thoracics * Chest pad * Padded box foot plate * Footplate pad * Knee blocks * Booties * Sandals & raising blocks

* Individual footplates * Small pad covers (for abduction block, hip pads, hip extension pads, lumbar roll, thoracics).


DIMENSIONS

	Size 1 (JUC01)	Size 2 (JUC02)	Size 3 (JUC03)
Age range (approx.)	18 months - 5 years	3 years - 8 years	5 years - 12 years
Seat depth	210mm/305mm	240mm/320mm	275mm/390mm
Seat width (min/max)	120mm/300mm	120mm/300mm	120mm/300mm
Lower leg length	175mm/280mm	240mm/380mm	240mm/420mm
Back height (min/max)	300mm/420mm	350mm/470mm	400mm/520mm
Seat height from floor	305mm/760mm	340mm/760mm	365mm/760mm
Tilt in space (supine)	25° supine - 15° prone	25° supine - 15° prone	25° supine - 15° prone
Max user weight/load	50kg	50kg	50kg
Base size (footprint)	780mm x 650mm	780mm x 650mm	780mm x 650mm


Jenx Junior is supplied with new reversible Contour Fabric seat and back pad covers in a choice of styles and colours. You choose the main colour (1), reverse colour (2) and quilt pattern (3).

1. MAIN COLOUR

CONTOUR 'PLUS' FABRIC

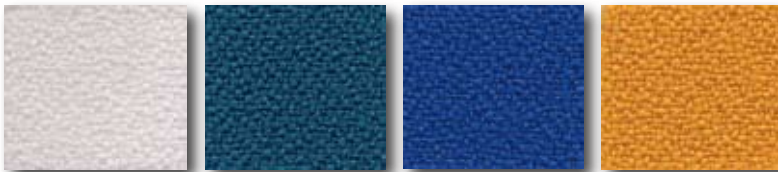

White Turquoise Bright Blue Mustard

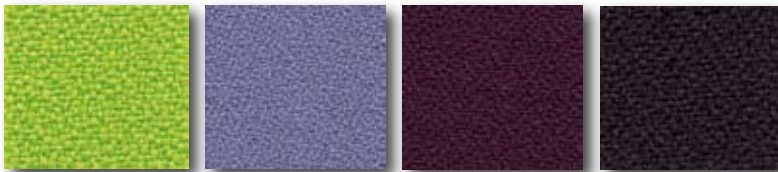

Lime Steel Purple Charcoal

CONTOUR 'WEAVE' FABRIC



Neutral Grey Cerise Really Red Petrol

CONTOUR 'SUPER SOFT' FABRIC



Stone Orange Pink Green

Colours supplied may vary from this guide.
Please ask to see actual swatch.

2. REVERSE COLOUR

PADDED SUEDE FABRIC



Black Light Blue




Grey Dark Blue

3. QUILT PATTERN



Standard (S)

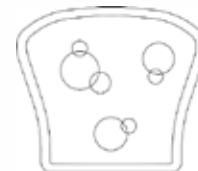

Bubbles (B)



Waves (W)

Hand finished quilt pattern may vary.
Standard quilt cover set: red with black reverse.
Reverse has matching coloured binding to edges.



Jenx Limited

Wardsend Road, Sheffield, S6 1RQ

Tel: 0114 285 3376

Fax: 0114 285 3528

sales@jenx.com

www.jenx.com

